

# Instruction Manual



**NO** Electricity  
Fumes  
Bad Smells

Compliance  
NABH | NABL

Introduction

**1**

Destroy Container

**4**

Installation

**2**

Operation

**5**

Sharp Container

**3**


Cleaning

**6**

# Know your Easy Cut & its Parts



## Understand Easy Cut & its Accessories

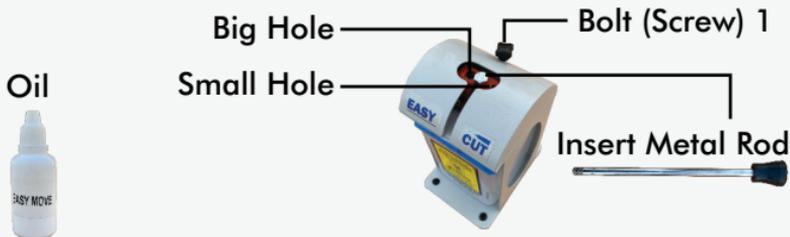

## Introduction

# 1

Easy Cut Needle Cutter is designed by a team of expert engineers & professionals considering the safety of BMW Workers.

Easy Cut is used to destroy almost all kinds of needle & syringe.

Easy Cut Needle Cutter compliance with the highest standards of NABL & NABH.

It does not require any electricity to operate hence does not produce any fumes or bad smell.

It can cut Syringe, Needle, Scalp Vein Set, Vacutainer Needle, Insuline Needle, Dialysis Needle, PFS Needle etc.

It is safe for Healthcare Professionals, BMW Workers as well as easy to use. It is maintenance free & can be installed anywhere.

Each Easy Cut comes with a Sharp Needle Container. This Sharp Container is made from high grade puncture resistant plastic material and it can be incinerated. The lid of container is designed to avoid any leakage during operation.

Ref No	Product
EC1	Easy Cut Needle Cutter
LSC15	Sharp Needle Container 1.5 Liter

## Installation

# 2

- Gently unpack the outer packaging & take out the Easy Cut unit by holding the outer box from bottom.
- You will find the Easy Cut unit, Sharp Container & a Metal Rod wrapped with a shrink wrapper.
- Slowly unwrap the unit & hold the metal rod in one hand. As soon as, unit is unwrapped, keep the metal rod aside.
- On top side, as shown in image, bolt 2 is fitted. This bolt is to be unscrewed & the metal rod is to be screwed up on the place of bolt 2. Whenever, the Easy Cut is to be kept & stand alone condition for a longer time or need to be repacked, just replace the metal rod with bolt 2.
- At the bottom of Easy Cut, there are rubber grip to avoid movement of unit when in use. To make sure it does not move at all, we highly recommend to keep the unit on non-slippery surface.
- The sharp container is already available with the unit. On top-back side of the unit, you can find bolt 1 as shown in image which is to be loose to take out the Sharp Container from the Easy Cut. If the bolt is tightly screwed up, the Sharp Container will not move.
- Now your unit is ready to use.



## Operation of Easy Cut

# 5

- After installation as per above guideline, your Easy Cut is ready to use.
- You can cut almost every kinds of needle or syringe.
- As shown in image, there are two holes, small hole and big hole.
- The big hole is used to cut almost all kinds of syringe & needle. However, small hole is to be used for Insuline Needle, Scalp Vein Needle and PFS Needle.
- Do not detach the needle from syringe.
- Insert the syringe along with the needle inside the respective hole. Move the metal rod towards down side gently to cut the syringe & needle. The cut part will be dropped into the sharp container.
- To cut Vacutainer Needle, first it is to be inserted into the small hole. Move the metal rod towards down side to cut the needle. Thereafter, take out the Vacutainer Needle from small hole & insert it into big hole. Move the metal rod slightly towards down side till the Vacutainer Needle is tightened up & hold the metal rod in that position. Thereafter, unscrew the Vacutainer Needle until the internal part of syringe is dropped into the sharp container.

## Cleaning

# 6

- ▶ Easy Cut is maintenance free, however cleaning is required.
- ▶ After usage of Easy Cut, you will find a few drops of blood as shown in image 1 here.
- ▶ To clean that part, just open the top flap of Easy Cut by unscrewing the bolt 1 marked in image 2.
- ▶ We recommend to clean the blood drops with IPA & Cotton Buds.
- ▶ After proper cleaning, just close the top flap & screw up as it was.

Image 1



Image 2





Regd. Office : 406, Shefali Shopping Centre, Paldi Char Rasta  
Ahmedabad-380006, Gujarat, India

Ph. : +91(079) 26578833

Works : Plot No. 776/1, Vadsar Ind. Area, Ta. Kalol  
Dist. Gandhinagar-382721, Gujarat, India

Ph. :+91(02764) 248137, Customer Care : + 91 7096028364

Email : [info@labtechdispo.com](mailto:info@labtechdispo.com)

Website : [www.labtechdispo.com](http://www.labtechdispo.com), [www.labtechdispo.in](http://www.labtechdispo.in)